


Deluxe Mobile Apron Rack

683407, 683408

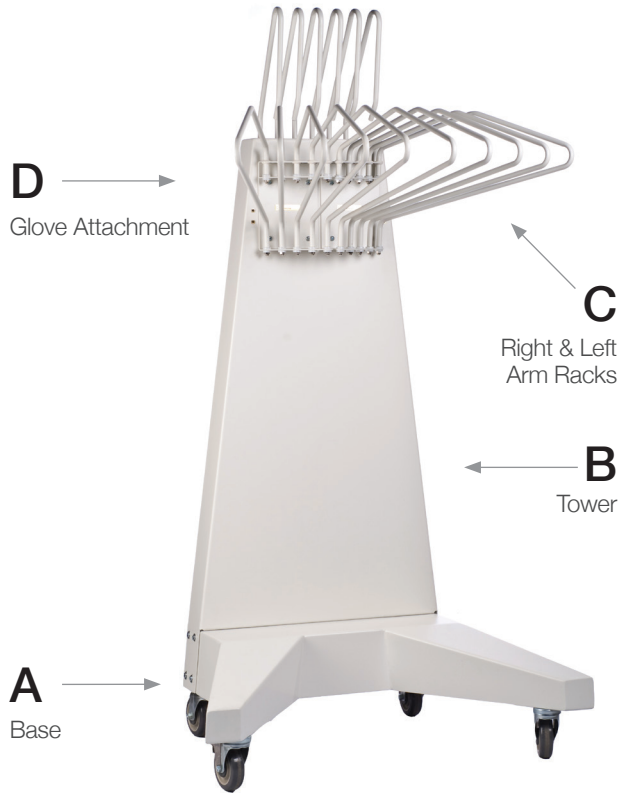

Step 1:

Set Base (A) on level surface.

Step 2:

Remove back plate on Tower(B) and place unit over corresponding holes on the Base(A). * Insert screws in all holes in the inside (5 screws) and outside (8 screws).

Step 3:

Place Right and Left Arm Racks (C) over corresponding holes on tower and tighten with the screws provided.

Step 4:

Set back plate back over Tower(B) and replace all screws. STEP 5 – (Optional) Place Glove Attachment (D) over the back plates top four holes and insert screws.

Model	Description	Box	Box Weight	Box Dimensions
683407	Deluxe Mobile Apron Rack Width: 27" / 68.58 cm Height: 62" / 157.48 cm Depth: 37" / 93.98 cm	Box 1	22 lbs. / 9.98 kg	16" x 32" x 13" / 40.64 cm x 81.28 cm x 33.02 cm
		Box 2:	53 lbs. / 24.04 kg	39" x 28" x 12" / 99.06 cm x 71.12 cm x 30.48 cm
		Box 3	70 lbs. / 31.75 kg	55" x 29" x 6" / 139.7 cm x 73.66 cm x 15.24 cm
683408	Mobile Apron Valet with 20 Hangers Width: 27" / 68.58 cm Height: 62" / 157.48 cm Depth: 37" / 93.98 cm	Box 1:	10 lbs. / 4.54 kg	5" x 20" x 26" / 12.7 cm x 50.8 cm x 66.04 cm
		Box 2	22 lbs. / 9.78 kg	16" x 32" x 13" / 40.64 cm x 81.28 cm x 33.02 cm
		Box 3:	53 lbs. / 24.04 kg	39" x 28" x 12" / 99.06 cm x 71.12 cm x 30.48 cm
		Box 4:	70 lbs. / 31.75 kg	55" x 29" x 6" / 139.7 cm x 73.66 cm x 15.24 cm