
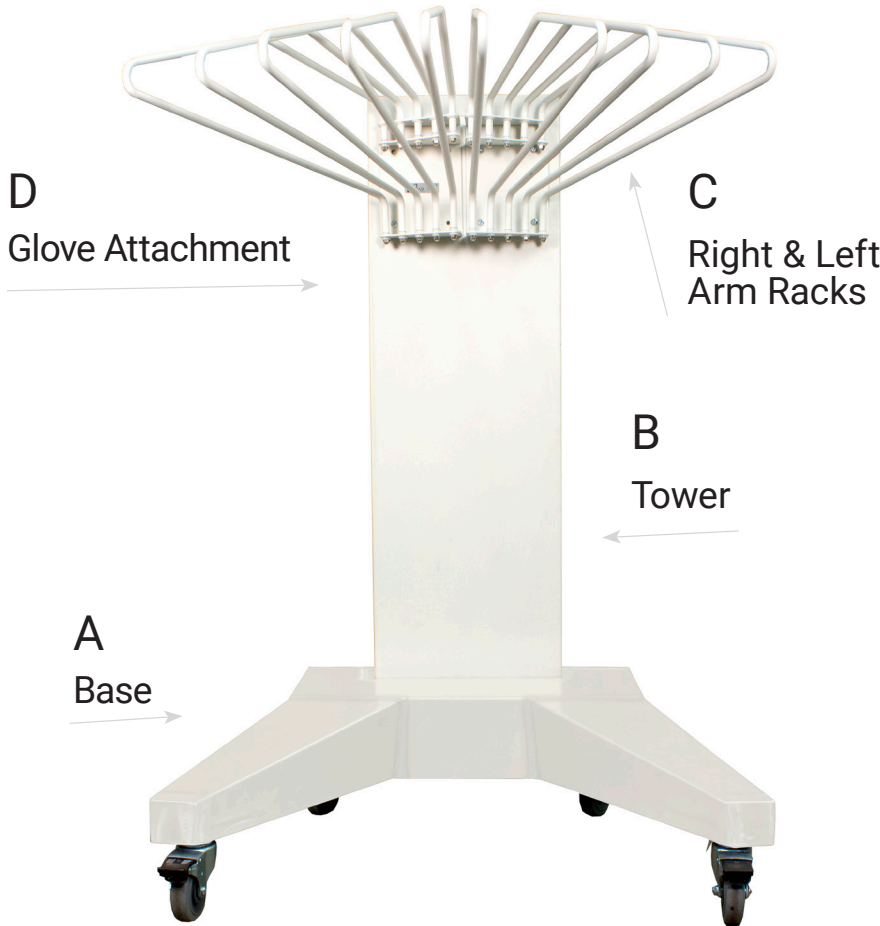


Deluxe Mobile Apron Rack
683407, 683408

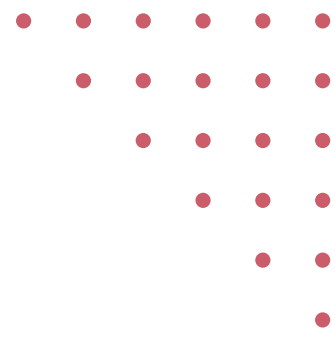

Step 1:

Set Base (A) on level surface.

Step 2:

Remove back plate on Tower(B) and place unit over corresponding holes on the Base(A). * Insert screws in all holes in the inside (5 screws) and outside (8 screws).

Step 3:

Place Right and Left Arm Racks (C) over corresponding holes on tower and tighten with the screws provided.

Step 4:

Set back plate back over Tower(B) and replace all screws. STEP 5 – (Optional) Place Glove Attachment (D) over the back plates top four holes and insert screws.

Model Number	Description
683407	Deluxe Mobile Apron Rack
Dimensions	53" H x 39" W X 26" D
Colors	White, Custom Options
Options	Powder Coated Steel Stainless Steel (Upgrade)
Wheels	4 swivel casters, 2 with brakes
Shipping	Box 1: 16" x 32" x 14", 25 lbs / 11.34 kg Box 2: 40" x 29" x 12", 55 lbs / 24.95 kg Box 3: 58" x 17" x 9", 52 lbs / 23.59 kg